Pregnabit[®] Pro

Mobile CTG

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
 <p>ISO 27001 / 27017 / 27018 ISO/IEC 27001:2013 Certification and ISMS relating to information security management systems for cloud services Find out more →</p>	 <p>ISO 27701 ISO/IEC 27701:2019 Certification and PIMS relating to personal data processing security management Find out more →</p>	 <p>GDPR Compliance with Regulation (EU) 2016/679, known as the General Data Protection Regulation (GDPR)</p>	 <p>CSA STAR Cloud Security Alliance (CSA) Level 1: Best Practices Self Assessment</p>
 <p>CISPE Founding member of Cloud Infrastructure Services Providers in Europe (CISPE) and signatory of its code of conduct</p>	 <p>ANSSI SecNumCloud French ANSSI certification for trusted cloud service providers Find out more →</p>	 <p>HDS Certification by the French Digital Health Agency (ANS) for hosting healthcare data Find out more →</p>	 <p>Healthcare data in Europe Compliance for hosting healthcare data from French, British, Italian, German and Polish citizens</p>
 <p>PCI DSS Lever 1 Payment Card Industry Data Security Standard certification relating to payment data hosting Find out more →</p>	 <p>ACPR Outsourcing Guidelines (France) Compliance for the provision of outsourced essential services from the French Prudential Supervision and Resolution Authority (ACPR) Find out more →</p>	 <p>SWIPO Signatory of the SWIPO IaaS Code of Conduct on cloud service portability</p>	 <p>C5 Cloud Computing Compliance Criteria Catalogue Find out more →</p>

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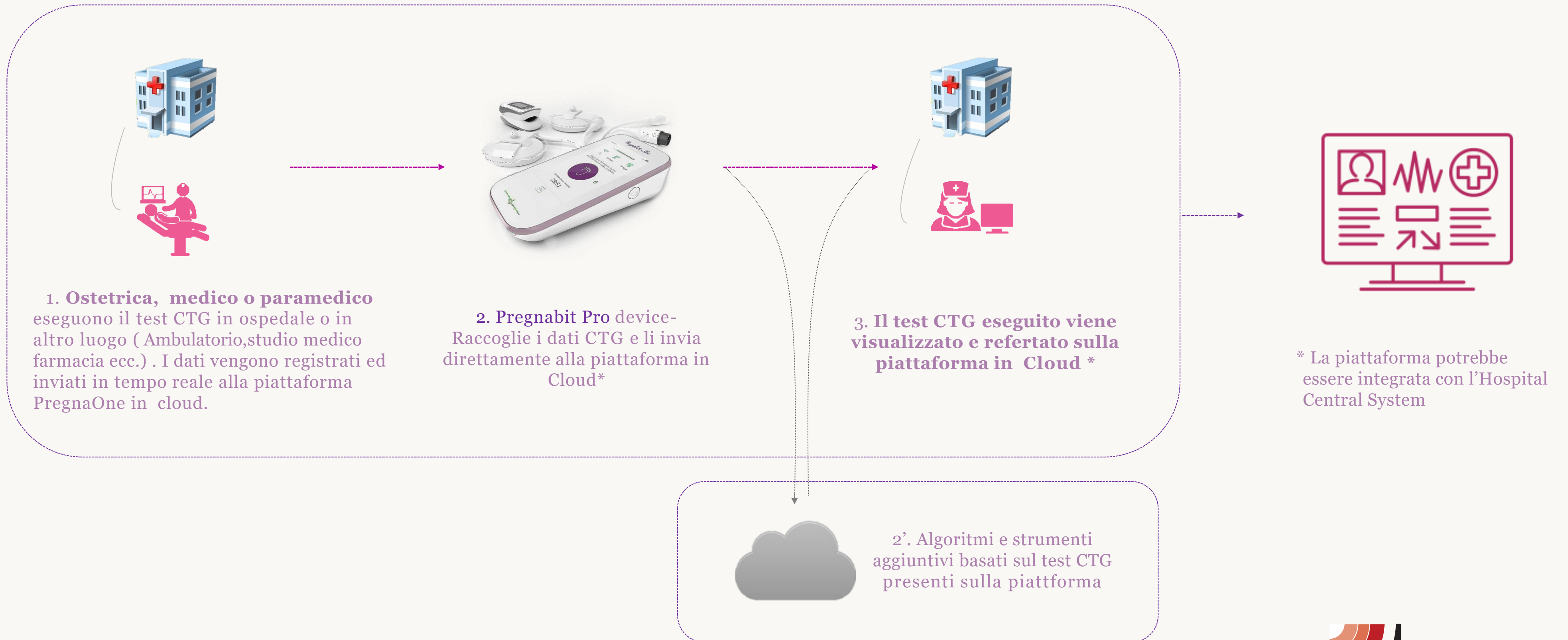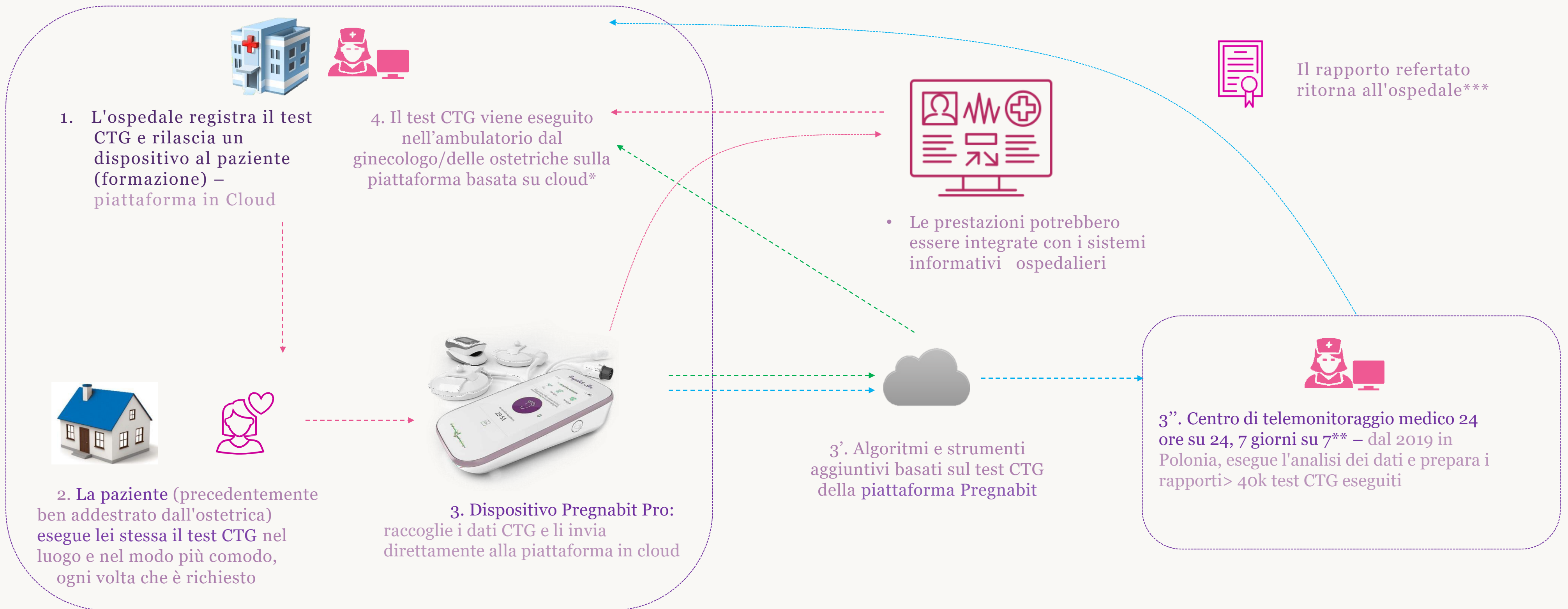
Home telemonitoring versus hospital care in complicated pregnancies in the Netherlands: a randomised, controlled non-inferiority trial (HoTeL)

Mireille N Bekker, Maria P H Koster, Willem R Keusters, Wessel Ganzevoort, Jiska M de Haan-Jebbink, Koen L Deurloo, Laura Seeber, David P van der Ham, Nicolaas P A Zuihoff, Geert W J Frederix, Josephus F M van den Heuvel, Arie Franx

Summary

Background Women with complicated pregnancies often require hospital admission. Telemonitoring at home is a promising alternative that fulfils a worldwide need in obstetric health care. Moreover, the COVID-19 pandemic has accelerated the transformation to digital care. The aim of this study was to evaluate safety, clinical effectiveness, patient satisfaction, and costs of home telemonitoring against hospital care in complicated pregnancies.

Methods We did a multicentre, randomised, controlled, non-inferiority trial in six hospitals (four general teaching hospitals and two university hospitals) in the Netherlands (located in Utrecht, Amsterdam, and Groningen). Women aged 18 years and older with singleton pregnancies (>26 weeks gestation) requiring monitoring for pre-eclampsia, fetal growth restriction, fetal anomaly, preterm rupture of membranes, reduced fetal movements, or history of fetal death were included in the study. Participants were randomly assigned to either hospital admission or telemonitoring in (1:1), stratified for the six diagnoses for inclusion and the six centres of inclusion, using block randomisation (block sizes of four and six). When assigned to telemonitoring, participants went home with devices for cardiotocography and blood pressure measurements and had daily contact with their care providers after digitally sending their home measurements. When assigned to hospital admission, participants received care as usual on the ward until the postpartum period. The primary outcome was a composite of adverse perinatal outcomes assessed after delivery, including mortality; an Apgar score below 7 after 5 min or an umbilical arterial pH at birth below 7.05; maternal morbidity; admission of the newborn to the neonatal intensive care unit; and rate of caesarean section. The primary outcome was assessed in the intention-to-treat population. The non-inferiority margin for the primary outcome was a 10% absolute increase in composite primary endpoint based on baseline 20% incidence. The study was registered at the Dutch Trial Registration (NL5888) and is now closed to new participants.

Lancet Digit Health 2023;
5: e116–24

See Comment page e103

Department of Obstetrics and Gynaecology (Prof M N Bekker PhD, J F M van den Heuvel PhD, Prof A Franx PhD); Department of Julius Center, Health Sciences (W R Keusters MSc, N P A Zuihoff PhD, G W J Frederix PhD); University Medical Center Utrecht, Utrecht, Netherlands; Department of Obstetrics and Gynaecology, Erasmus Medical Centre, Rotterdam, Netherlands (M P H Koster PhD, Prof A Franx); Department of Obstetrics and Gynaecology, Amsterdam University Medical Center, Amsterdam, Netherlands (W Ganzevoort PhD); Department of Obstetrics

Diagnostic equivalency of mobile CTG devices and remote analysis to conventional on-site nonstress test

Renata Pilarczyk^{1,A–D}, Mateusz Strózik^{2,1,A–D}, Lidia Hirnle^{1,E,F}

¹ 1st Department and Clinic of Gynecology and Obstetrics, Wrocław Medical University, Poland

² Division of Histology and Embryology, Department of Human Morphology and Embryology, Wrocław Medical University, Poland

A – research concept and design; B – collection and/or assembly of data; C – data analysis and interpretation; D – writing the article; E – critical revision of the article; F – final approval of the article

Advances in Clinical and Experimental Medicine, ISSN 1899–5276 (print), ISSN 2451–2680 (online)

Adv Clin Exp Med. 2020;29(1):33–44

Address for correspondence

Mateusz Strózik
E-mail: juriaa@gmail.com

Funding sources

None declared

Conflict of interest

None declared

Received on February 28, 2019

Reviewed on June 15, 2019

Accepted on August 18, 2019

Published online on January 21, 2020

Abstract

Background. Remote pregnancy monitoring is one of the most promising applications of telemedicine; however, the diagnostic value of self-examination using mobile cardiotocography (CTG) devices and remote analysis of the subsequent results has never been properly studied.

Objectives. The study aimed to compare the diagnostic usefulness of CTG self-examination using a mobile device to examination performed by a medical professional using a stationary device; and to evaluate the quality of CTG analysis performed remotely.

Material and methods. Eighty-two pairs of CTG recordings were collected; each pair consisted of a single recording from an examination performed by a midwife using a stationary device, and another recording from an unassisted patient self-examination using a mobile device. Recordings were performed with a maximum time interval of 30 min. Each recording was analyzed twice. Primary analysis included a comparison of the assisted examination evaluated on-site vs the self-examination evaluated remotely in pairs. Secondary analysis was conducted by an independent expert who evaluated the unpaired recordings. Baseline fetal heart rate (BFHR) values were compared independently.

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